PATIENT USE AND CARE INSTRUCTIONS

CONTENTS

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Patient Use and Care Instructions

CONTENTS

What is Angel Aligner?	02
Read Before Use	02
- Indications for use	02
- Contraindications	02
- Patient Instructions	02
- Safety Considerations	03
Wearing Angel Aligners	04
- Preparation before aligner insertion	04
- Aligner Insertion	04
- Aligner Removal	05
- Attention	06
Care and Maintenance	07
- Aligner Storage & Cleaning	07
- Aligner Wear Time	07
- Office Visit Requirement	08
Q&A	08
Glossary	10

WHAT IS ANGEL ALIGNER?

The clear aligner system consists of a series of doctor-prescribed, BPA free, thin, clear plastic removable orthodontic appliances (aligners) and proprietary 3D software. The aligners gently move the patient's teeth in small increments from their original state to a more optimal, treated state.

READ BEFORE USE

Before using Angel Aligners, patients (and parents or legal guardians) should carefully read these Use and Care Instructions for important information about aligner wear, including warnings and precautions, proper care and maintenance, and oral hygiene requirements during treatment. We recommend that you save these instructions for future reference. If you have questions or concerns, please contact your doctor.

Indications for use

Indicated for the alignment of teeth through orthodontic treatment of malocclusion.

Contraindications

Angel Aligners are contraindicated for patients with: Serious progressive periodontal disease, or known TPU/PETG/PE allergies.

Note: Patients using Angel Aligners should be able to understand and follow any treatment instructions as provided by doctor.

Patient Instructions

- Aligners are intended to be worn 22 hours or more per day.
- After eating or drinking sugary drinks, patient should take oral cleaning measures (such as gargling with water, brushing their teeth, etc.) before wearing aligners.
- Patient should not use hot or disinfecting water to soak the aligners to avoid deformation. Do not use denture cleaners or soak in mouthwash to clean aligners.
- Patient should contact the doctor as soon as possible for the excessive irritation to the lining of the mouth caused by wearing the aligners, and no self-adjustment by patient shall be allowed.
- In the case of gum injury, oral ulcers and abnormal loosening of teeth caused by the appliance itself, patient should contact the doctor as soon as possible. If necessary, stop using the orthodontic aligners and wait for the tissue to recover.
- Failure to wear the aligners for the prescribed number of hours per day and/or not using the product as directed by the doctor can lengthen the treatment time and affect the ability to achieve the desired results.
- When the aligners are damaged, deformed or lost, patient should contact the doctor as soon as possible.
- Tobacco use, smoking, and/or vaping while wearing aligners may lead to discoloration or damage of the aligners.

Safety Considerations


- a. The aligners shall be used under the guidance of the attending doctor, and the treatment plan for its use shall be formulated by the attending doctor.
- b. The aligners are made for a particular patient, at a particular stage and in a correct sequence. The aligners must be used in order determined by the orthodontic sequence and can only be used for that particular patient.
- c. In the use of the aligners for orthodontic treatment, the patient may experience a certain level of discomfort. This is considered normal. If there are questions or prolonged discomfort, the patient should contact the doctor immediately.
- d. If there is discomfort when wearing the aligners, patient should not adjust the aligners, but contact the doctor as soon as possible.
- e. After receiving the aligners, the patient shall wear immediately to prevent unintended tooth movement.
- f. If the patient has one or more missing teeth, he or she must wear retaining aligners immediately.
- g. If the patient is unable to wear Angel Aligner for the recommended time, does not comply with the doctor's instructions for wear, cannot complete return visits on time, there is the presence of abnormal shape of teeth, or there is new tooth growth during treatment, orthodontic treatment and timing may be affected.
- h. Some medical conditions and the use of drugs may affect the treatment.
- i. Malocclusion, severe overbite, and mixed dentition may require treatment other than Angel Aligner.
- j. The combination of other fixed appliances/active orthodontic aligners may be required for complex treatment options that are not fully competent with the Angel Aligner.
- k. Severe dentition congestion or occlusal imbalance may require the intervention of oral surgery. If this type of surgery is needed, the corresponding risks should be considered before treatment.
- l. Unconventional shapes, sprouting or missing teeth may affect the wearing of orthodontic aligners or may affect the treatment.
- m. A tooth with fillings or restorations may be displaced, and the fillings or restorations may need to be replaced.
- n. The aligners do not move dental implants effectively.
- o. In cases of traction accessories, the patients are required to cooperate with the traction according to the doctor's instructions.
- p. Patient should avoid improper intraoral habits, improper usage, etc. If the aligner is fractured or damaged, or the surface of the aligner is peeled, patient should contact doctor immediately.
- q. If the aligner cannot be engaged, contact the doctor immediately.
- r. In case of damage to the lining of the mouth caused by wearing the aligners, the patient should contact the doctor as soon as possible and no self-adjustment shall be allowed.
- s. In the case of gum injury, oral ulcers and abnormal loosening of teeth caused by the non-aligners itself, patient should contact the doctor. If necessary, stop using the orthodontic aligners and wait for the tissue to recover before continuing to correct.
- t. When the aligners are damaged, deformed or lost, patient should contact the doctor and take prompt remedial measures.
- u. In rare instances, problems in the temporo-mandibular or jaw joint temporo-mandibular disorder or dysfunction (TMD) may result in joint pain, headaches, or ear problems. For patients in treatment with mandibular advancement features such as Occlusal Block (which are used to shift the lower jaw forward), problems in the jaw joint may be exacerbated.

WEARING ANGEL ALIGNER


• Preparation Before Aligner Insertion

1. Check the label on the back of the packaging prior to inserting the new aligner. (Figure 1)
2. Wash your hands before touching the aligner.
3. Use a soft bristled toothbrush to gently clean the inside and outside surface of the aligner using cold water or mouthwash. Then rinse the aligner with water. Do not use hot water or disinfectant to clean the aligner as this may lead to deformation of the aligner.
4. Brush your teeth and rinse with water before inserting aligners – after eating or drinking.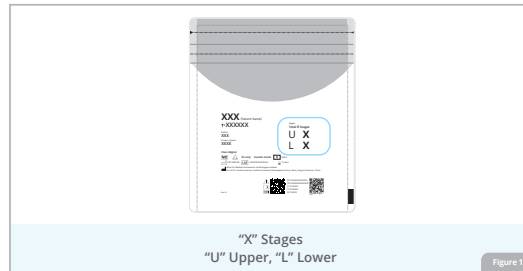

• Aligner Insertion:

1. You may insert either the upper or lower aligner first. Insert the aligner by positioning the aligner over the front teeth first. Once the aligner is in place, use gentle pressure and guide the aligner over the teeth moving towards the back until the aligner clicks into place. (Figure 3)
If using A6, your doctor will help ensure the blocks are positioned correctly the first time. The blocks should exert gentle pressure on the jaw during biting and talking.

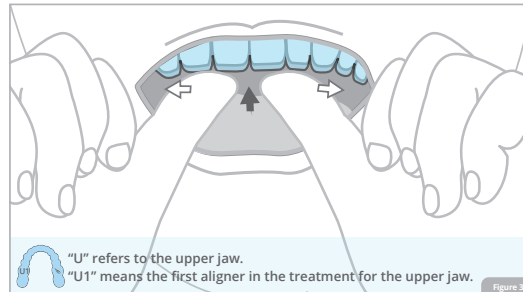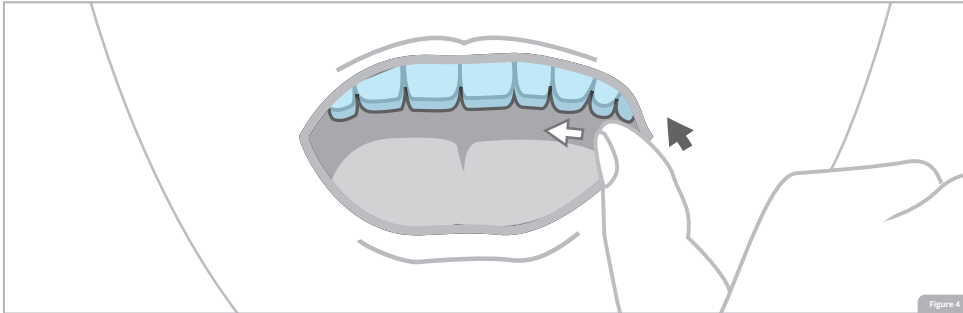
Figure 4

2. After pressing, check to make sure the bite between the upper and lower jaw is fine and that the aligner fits on all the teeth. (Figure 4)
3. If the aligners do not seat completely, use the chewies provided in your Starter Kit to help guide them into place. If aligners do not fit at all, consult your doctor for further instruction.

• Upper Aligner Removal:

Starting at the back of the mouth, use your finger to pull the inside edge of the aligner gently away from the teeth. Then alternate pulling between left and right side of the upper aligner until it is removed. (Figure 5)


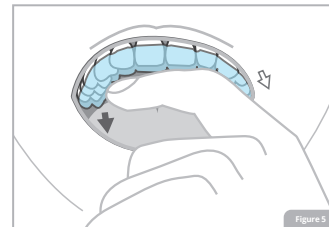
Figure 5

- **Lower Aligner Removal:**

Starting at the back of the mouth, use your finger to push the outside edge of the aligner gently away from your teeth. Then alternate pushing between the left and right sides of the aligner until it is removed. (Figure 6)

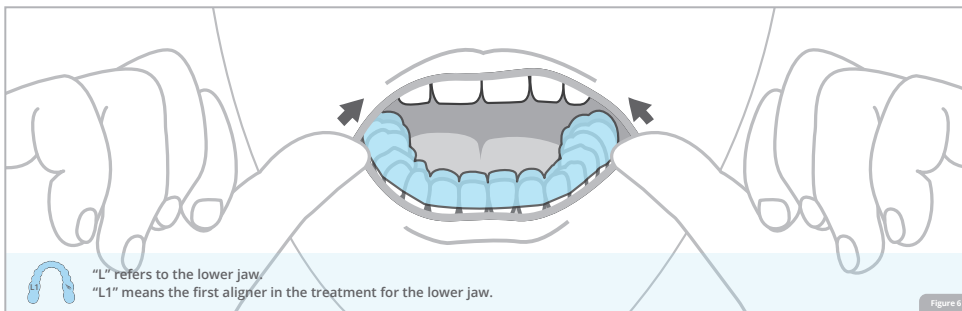- **Attention:**

1. At the beginning of wearing new aligners, you may feel slight discomfort or soreness. This is a normal reaction of your teeth to orthodontic treatment. If you feel sharp pain or discomfort, please contact your doctor.
2. If you have difficulty inserting or removing your aligner, you can use your nail to gently lift the edges of the aligner in the area where it's hard to engage or remove. Then gently press the aligner to fit or wiggle slowly to remove. Do not use too much force, as this can cause the aligner material to fracture or distort.
3. Prior to inserting your aligners, inspect them for cracks or deformities. If your aligner is damaged, distorted, or lost, please contact your doctor as soon as possible for a replacement and for further direction to avoid excessive treatment delay.
4. If your gingiva (gum tissue) becomes irritated from the aligners, do not try to modify the aligners. Contact the doctor as soon as possible for further instruction. If during treatment, you experience gingiva damage, dental abscess or loosening of teeth, contact the doctor as soon as possible for further instruction. If necessary, put treatment on hold and resume when fully recovered.

CARE AND MAINTENANCE

• Aligner Storage and Cleaning

1. Aligner storage case(s) are included in your Starter Kit. We recommend that you store your Angel Aligners in a case when they are not in your mouth. This will help protect them from loss and damage. Keep aligner cases clean by hand washing them with soap and water.
2. If aligners need to be removed to eat or drink, please rinse your aligners and place them in the provided case for safe keeping. (Figure 7)

• Aligner Wear Time:

1. Wear your aligners for 22 hours or more per day or as directed by your doctor.
2. Each set of aligners is designed to be exchanged with the next stage aligner after at least every 10 days or as instructed by your doctor.
3. If for any reason you are not able to wear your aligners for the recommended 22 hours (or as directed by your doctor) you may need to extend the number of days, the aligner stage is worn. Consult your doctor for further instruction.

- **Office Visit:**

Office visits are often necessary for effective treatment and progress tracking. If you miss an appointment, call to reschedule as soon as possible.

Wear your aligners when you visit your doctor and bring your last 2 stages of aligners with you to your appointments unless otherwise directed by your doctor.

Q&A

Q: Why am I experiencing soreness?

A: Many patients will feel soreness for the first couple of days after starting each new aligner stage. This is completely normal. This indicates that the aligner is applying force to your teeth and moving your teeth towards the desired position. This force is very light and will not cause any damage to your teeth. The soreness will generally reduce after wearing for a few days.

Q: Are there restrictions on what I can eat?

A: In general, no. Unlike traditional orthodontics, you can usually eat and drink whatever you desire because you remove your aligners while eating. Thus, there is no need to restrict your consumption of any of your favorite foods and snacks, unless otherwise instructed by your doctor.

Q: When wearing aligners, can I drink hot or cold drinks?

A: Drinking anything besides cool water while wearing aligners is not recommended. Keep aligners away from alcohol and hot, sweet, or colored liquids which can damage the aligner and cause cavities or white spot lesions on teeth.

Q: Will Angel Aligners affect my speech?

A: Like any other orthodontic treatment, our customized aligners may affect your speech for the first 1-2 days. Your speech will typically return to normal once you become adjusted to it.

Q: As the aligner is made of plastic, will it be damaged easily?

A: Angel Aligners are made of clear, durable, and medical grade plastic. While the aligners are strong and flexible, ensure that you are using them correctly, in accordance with the use instructions and your doctor's directions.

Q: What do I do if I lose or damage my aligner?

A: If you lose or damage your aligner, please contact your doctor as soon as possible. Your doctor is likely to tell you to move back to your last stage of aligners – this is why it is very important to keep your last 2 stages of aligners.

Q: For patients undergoing mandibular advancement, why do some of my aligners have 'blocks' on them?

A: The 'blocks' on the sides of aligners are designed to position the lower jaw forward to help correct the bite.

Q: Why do some of my aligners have bumps or ridges on them?

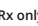
A: Depending on your specific treatment, some of the movements may require either "attachments" or "ridges" to help the aligner grip the teeth. These bumps, or wells, are where the aligner grips the attachment the doctor places on your teeth. The attachments are actually small pieces of composite the doctor affixes to your teeth that are then gripped by bumps on your aligners. The ridges are slim indentations in your aligners. Your doctor will use one or both of these features to attempt to achieve the desired movement.

Q: What if I lose or break an attachment?

A: In the event that an attachment is lost or broken, you should immediately contact your doctor.

Note: If you have additional questions after reading this pamphlet, please consult with your doctor.

Glossary

Symbol	Description	Symbol	Description
	Batch Code		Use-by Date
	Caution		Consult instructions for use or consult electronic instructions for use
	Medical Device		Manufacture Date
	Manufacturer Name and Address		Prescription Use Only
	Model Number		Non Sterile
	Patient Number		Unique Device Identifier

Manufacturer: Wuxi EA Medical Instruments Technologies Limited.

Registered address: No.1619, Huishan Avenue, Huishan Economic Development Zone, Wuxi, Jiangsu Province, China.

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Fabricante: Wuxi EA Medical Instruments Technologies Limited.